



SagiNova®

Reliability, Safety, and Performance in HDR Brachytherapy

SagiNova®

The SagiNova® HDR afterloader was designed to address today's needs for HDR brachytherapy applications regarding safety, seamless workflows, and cost efficiency.



SagiNova's world-class technology is based on our innovative approach to incorporate customer needs into product development.

Miniaturized Iridium-192 and Cobalt-60 Source

Eckert & Ziegler BEBIG provides the choice of two isotopes, Ir-192 and Co-60, according to individual institutional needs and preferences. Both sources are equivalent in size and have been in clinical use for more than 10 years. Due to the considerably longer half-life of five years and a certified 100,000 source transfers, more and more institutes choose the Co-60 source as it requires time to perform QA procedures related to source exchanges only every five years.

Integrated Real-Time In-Vivo Dosimetry System



The unique In-Vivo Dosimetry System allows direct independent monitoring of doses to rectum and bladder at the control console alongside treatment information. Dose limit values can be defined via the SagiNova® treatment control software and warnings are displayed if bladder or rectum dose limits are exceeded. The complete data is integrated in the treatment report for a convenient documentation. Since the system is operated directly from the SagiNova® treatment control station, In-Vivo Dosimetry is easily integrated into the treatment process with no additional equipment or screens.

QAssist – Guidance in Quality Assurance



The highly customizable tool QAssist supports physicists by defining and running site specific quality assurance protocols. Guiding the performance of these protocols, QAssist ensures streamlined and well documented quality assurance.

Automatic Length Measurement



By choosing an applicator in SagiPlan®, the length of the applicator and the overall length of the application are digitally transferred to the afterloader without requiring any manual input. Hence, the afterloader is able to validate the correct overall length per channel prior to each source movement. SagiNova® measures the internal length of the catheter or applicator allowing the user to review and complete acceptance while the unit completes a verification of the length before the treatment is delivered. The ultimate benefits of the Automatic Length Measurement are improved patient safety and peace of mind for clinicians when treating body sites such as head & neck, breast, and skin with flexible catheters or breast balloons.

Optimized User-Friendly GUI and Intuitive Design for Streamlined Workflow

The new SagiNova® graphical user interface (GUI) was created in cooperation with brachytherapy experts and usability engineers leading to the development of the next level of smooth and efficient workflow guidance.

50 Channel Support

As brachytherapy has evolved to more sophisticated applications, Eckert & Ziegler BEBIG provides an afterloader to support even the most complex treatments with up to 50 channels.

Round-the-Clock Service Excellence

The Eckert & Ziegler BEBIG technical support, including real-time remote desktop assistance, is available globally to ensure a timely response and optimal clinic uptime. To meet individual institutional needs, tailor-made service and support plans as well as education and training programs are available.



Eckert & Ziegler
Contributing to saving lives

Main Features

- Ability to choose between widely used Ir-192 or long-lasting Co-60 source
- DICOM data import from brachytherapy planning system via network or USB flash drive
- Display and control of entire treatment process
- Recording and DICOM export of entire treatment protocol
- Synchronization of electronic timers in control and treatment unit with better than 50 ms accuracy
- Support of up to 100 dwell points for treatment length of 600 mm per channel (1400 mm drive-out length)
- Minimum stepping distance of 1 mm
- Source positioning accuracy of ± 1 mm and a resolution of 0.1 mm and 10 ms
- Continuation of interrupted treatments e.g. after an external power failure
- Support of up to 50 channels per fraction
- Optimized user-friendly GUI and intuitive design for streamlined workflow


Safety Features

- Highly customizable quality assurance system QAssist
- Video camera for source position verification (video and photo)
- Automatic length verification for precise measurement of applicator length
- Color coding for afterloader channels, applicators, and transfer tubes
- Battery backup system for safe source retraction in the event of a power failure
- Hand crank for power independent emergency source retraction
- Two independent monitoring systems for treatment times
- Integrated radiation detector
- Two-key authorization on treatment control panel
- User management incl. role concept and individual passwords
- Display of patient photo from treatment plan
- Internal shielding container for storage of up to 81.4 GBq Co-60 or 481 GBq Ir-192

Technical Data


Dimensions	(W x D x H) 552 x 654 x 1000 mm
Weight	Co-60 / Ir-192 version 267 kg Ir-192 version 155 kg
Voltage range	100V-240V, 50/60 Hz, 1/N/PE
Classification	Class I, Type B equipment according to IEC 60601-1, Section 6
Dose rate at 5 cm distance	For Co-60 < 0.1 mSv/h For Ir-192 < 0.01 mSv/h Radiation shielding for both sources fulfill IEC 60601-2-17
Radioactive sources	≤ 81.4 GBq Co-60 $\varnothing 1.0$ mm ≤ 481 GBq Ir-192 $\varnothing 0.9$ mm
Active source component	$\varnothing 0.5 \times 3.5$ mm Co-60 $\varnothing 0.6 \times 3.5$ mm Ir-192

Automatic Length Measurement

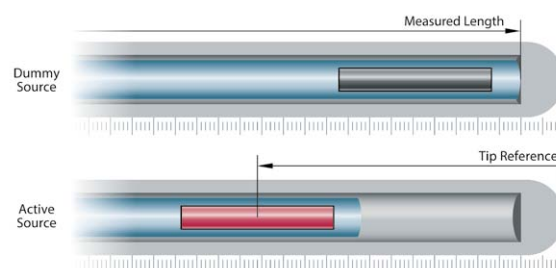

The dummy source measures the length of the complete channel prior to each source movement to allow for the verification of the correct channel length. The precisely measured length of each individual channel is automatically used for an accurate positioning of the active source relative to the tip reference.

Anisotropy

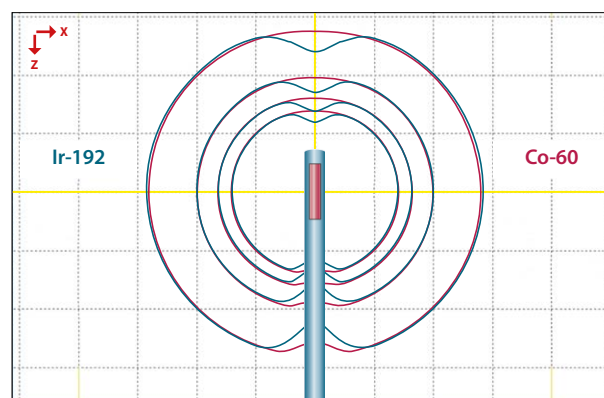

Anisotropy of Co-60 (red lines) and Ir-192 (blue lines)

The mentioned products are not available in all markets. Please contact your local Eckert & Ziegler BEBIG representative for more information.

Corporate Head Office:

**Eckert & Ziegler
BEBIG s.a.**
Rue Jules Bordet
Zone Industrielle C
7180 Seneffe
Belgium

Phone +32 64 520 811
Fax +32 64 520 801
info@bebig.com

Manufacturer:

**Eckert & Ziegler
BEBIG GmbH**
Robert-Rössle-Str. 10
13125 Berlin
Germany

Phone +49 30 94 10 84 130
Fax +49 30 94 10 84 112
info@bebig.com

Regional Sales, Marketing and Service:

Europe, Middle East, Africa,
Latin America, Asia Pacific

**Eckert & Ziegler
BEBIG s.a.**
Rue Jules Bordet
Zone Industrielle C
7180 Seneffe
Belgium

Phone +32 64 520 811
Fax +32 64 520 801
info@bebig.com

North America

**Mick Radio-Nuclear Instruments, Inc.
An Eckert & Ziegler BEBIG Company**
521 Homestead Avenue

Mount Vernon, NY 10550
USA

Phone +1 914 667 3999
Fax +1 914 665 8834
sales@micknuclear.com

www.saginnova.info
www.bebig.com
www.micknuclear.com